# Rocanal Permanent Gangrene R3

FR

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# Rocanal Permanent Gangrene R3

GB

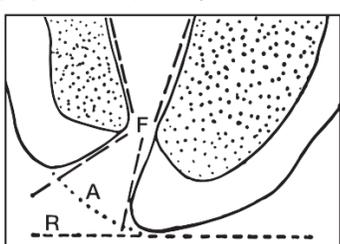
A permanent root canal filling cement for infected root canals

## REFILL / COMPOSITION

### Powder

Zinc oxide,  
Titanium oxide,  
Zinc stearate,  
Zinc acetate,  
Orthophenylphenol (OPP),  
Calcium wolframate,  
Anise

to prepare just before use (see mixing instructions hereafter)

F = physiological foramen  
A = anatomical foramen  
R = radiological foramen

Root canal filling after treatment of a non vital tooth (reaches the anatomical foramen or goes slightly over)

**WARNING: This medical device must be used only with the liquid of Rocanal Permanent Gangrene R3 (REF 0471).**

### INDICATIONS

A permanent sealing of infected root canals to be used after complete preparation of gangrenous canals, for adolescents over 12 years of age (at least 30 kg) and adults. It is advisable to treat one tooth per day with a maximum of 5 treatments per year for adolescents over 12 years and 10 treatments per year for adults.

### CONTRA INDICATIONS / SIDE EFFECTS

Not indicated in case of intolerance or allergy to one of the components and in particular in case of allergy or suspected allergy to Eugenol. A temporary periapical parodontitis may appear after applying the product and resolves in two days with proper treatment. No side effects have been reported to date.

### DIFINITION / REMARKS:

The permanent root canal filling is the final step in the treatment of a gangrenous tooth. The gangrenous tooth, on the contrary of a vital one, has the root canal completely infected up to the apical region. The defense against this bacterial action is in the well irrigated periodontium. In the canal the natural barrier is the limit between the enamel and the primary cement. **Diagnosis:** X-ray-Symptoms-objective examination. **Asymptomatic (chronic) cases:** tooth is necrotic -presence of large fillings- fistula- history of an acute anterior episode-negative response to pulp-tester; etc. **Acute cases:** periodontal pain-periapical abscess, periostitis, sub mucous abscess, etc.

### GENERAL PRINCIPLES

- Isolate the tooth with a coffer dam.
  - The root canal should never be left open.
  - Avoid partial root canal preparations and partial intracanal medications, because it may easily cause infection or acute periapical reaction. Canal treatment and cleaning must be completed in the same session.
  - Control always if there are deep gingival pockets.
- The difference between the treatment of a vital tooth and a non vital one, is that for the first the operator follows the surgical principle of "open and close immediately" in the same session if possible. For the non vital/infected case, it is necessary to proceed with a particular preparation of both the canal and the periapical region before performing the permanent filling of the canal (which will be performed in a subsequent session).

### TREATMENT SYSTEM

#### First session

1. Immediate and complete cleaning of the infected canal as far as the apex, by means of mechanical and chemical action (antiseptic). It is recommended to thoroughly irrigate with ROCANAL IRRIGATION, rinse and dry.
2. Preventive drug prescription for the periapical region if necessary.

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Erste Markenhinterlegung CE: 1998 REV: 05/2021 0476

Second session

Before filling permanently, it is essential to shape, clean and lubricate with ROCANAL IRRIGATION or the product of your choice. Final root-canal filling as far as the apex (anatomic foramen) use ROCANAL PERMANENT GANGRENE R3. ROCANAL PERMANENT GANGRENE R3 has especially been developed to meet the requirements of a filling material to be used after treatment of non vital teeth.

### Action of ROCANAL PERMANENT GANGRENE R3

Action studied in function of unextracted and infected pulp remnants. ROCANAL PERMANENT GANGRENE R3 is active over two distinct phases: the soft phase and after hardening. It is not resorbed in the canal system.

### PRODUCT INSTRUCTIONS:

1. Use sterile gloves and a sterile mixing slab and two single-end flexible sterile spatulas (one for powder, one for mixing).
2. Put one or two drops of liquid and small quantity of powder on the slab.
3. Amalgamate powder slowly with the liquid, with some pressure on the spatula.
4. Mix as long as needed to obtain a creamy, crystal-free mixture (crystals present in the powder are soluble in the liquid).
5. Proceed with your usual filling technique.
6. Use sterile material.
7. The setting time will depend on the consistency desired by the practitioner, depending on the powder/liquid ratio and the temperature. The average time of hardening of the mixture is 3 to 4 minutes.

### PRECAUTIONS FOR USE

For professional use only. Eliminate balances of unused product with the official rules in force.

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